

Knee Brace User Manual

Warning & Safety Information

- ⚠️ Do not wear during intense physical activity to avoid injury.
- ⚠️ Individuals with sensitive skin are advised to perform a patch test prior to use.
- ⚠️ Discontinue use immediately if skin irritation occurs.

II. Product Name

Knee Brace

III. Brand

AOLIKES

IV. Instructions for Use

1. Slide the knee brace onto the upper part of the knee and adjust the elastic strap for comfort.
2. Ensure the brace fits snugly around the knee joint to prevent slippage.
3. Maintain stability of the knee brace during movement.

V. Precautions

1. Avoid prolonged wear; remove and rest every 2 hours.
2. Not machine washable—hand wash only and air dry in shade.
3. Do not store with sharp objects.

VI. Features

1. Thermal Insulation: Constructed with thickened material for effective warmth retention.
2. Support & Protection: Provides targeted knee support to reduce strain and fatigue.
3. Adjustable Fit: Elastic, customizable design accommodates various leg shapes and sizes.

VII. Frequently Asked Questions (FAQ)

- Q: What should I do if the knee brace slips down?
A: Adjust the elastic strap or select a different size.

VIII. After-Sales Service

For quality-related issues, please retain your purchase receipt and contact customer service.

